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Problème	Possible cause	Solution
Failure of sprayer function	Batteries are low or improperly installed. Mesh sheet of the sprayer head is clogged. Sprayer head isn't properly installed.	Check or replace the batteries. Remove the sprayer head and reinstall it.
If the above solutions still do not solve the problem, please contact Bebeconfort customer care.		

G - EMC TABLES

NS13A is intended for use in the electromagnetic environment specified below. The customer or the user of NS13A must make sure that it is used in such an environment.
Guidance and manufacturer's declaration - Electromagnetic emissions

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A - PRODUCT FEATURES

1. Newborn silicone tip
2. Toddler silicone tip (2 years)
3. Collection cup
4. Sprayer button
5. Suction button
6. Sprayer water tank
7. Battery compartment
8. Spray head lid
9. Mist spout

B - BATTERY INSTALLATION

- Slide the battery cover off.
- Insert or replace 2 x 1.5 V AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols.
- Replace the battery cover.

Note:

- Always use NEW batteries; never mix old and new batteries.

• It is recommended not to use rechargeable, unqualified or different spec of batteries as it may damage the device or cause circuit shortcut.

• Please dispose of batteries properly and keep away from children and heat sources.

• If the device will not be used for a long time, it is recommended to remove the battery.

• Batteries must be disposed of in accordance with local environmental and institutional policies.

• Please replace all batteries when there is a decrease in suction or when battery power fails to drive the device.

C - INSTRUCTIONS FOR USE

How to use the sprayer function

1. Before use, remove the sprayer head.
2. Open the sprayer head lid and pour sterile water or saline into the tank. Add water carefully to "MAX" mark to avoid overfilling the tank.
3. Attach the sprayer header to the device. Check that the sprayer head and body of the device are securely attached with no gaps.
4. Place the sprayer head about 2 cm away from the nostrils, press the sprayer button to start spray function.
5. Press the sprayer button again to stop spray function any time as you need. Otherwise the spray function will automatically stop after 10 seconds.

Note:

- Use sterile water or saline only.

The amount of water in one spray is 0.4mL.

How to use the suction function

1. Choose the appropriate tip according to the situation: newborn or toddler tip.
2. Install the silicon tip and collection cup properly to the device.

3. Press the suction button to activate the suction function and ensure it is operating correctly before placing into the nostril.

4. Place the silicone tip into children's nostril and use suction to remove mucus.

Repeat in the other nostril as needed. If the children feel uncomfortable (crying, screaming, difficulty breathing, intense reactions, apnea, cyanosis, etc), please stop using it immediately.

5. Press the suction button again to stop the suction function or the device will automatically stop after one minute for power saving. The appropriate suction time for each procedure is between 5 to 8 seconds.

Note:

- Be careful not to place the tip too far into the nostril or press against the inner wall of the nostril.

• Excessive use of this device may result in pain, mucosal injury or respiratory distress in some circumstances.

D - CLEANING AND DISINFECTION

The following cleaning procedures should be performed after each use. Failure to clean equipment properly can cause clogs and shorten product life. The device should not be reused until thoroughly cleaned.

Clean the sprayer head

1. Remove the sprayer head lid and pour out any water remaining in the sprayer head.
2. Pour water into the cup, cover with sprayer head lid, shake several times and pour out the water.

3. Let all parts dry completely before reassembling.

Clean the silicon tips and collection cup

1. Carefully separate the silicon tip and collection cup from the device.

2. Cleaning: Wash the silicon tips and collection cup with mild, non-abrasive soap and water.

3. Disinfecting: Rinse the silicon tips and collection cup with 70°C water.

4. Let all parts dry completely before reassembling.

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